

Male incontinence product

Abri-Man Premium




Product features


- Protection designed specifically and only for male users.
- Secure and discrete fit into normal underwear.
- Pockets and a unique barrier system to prevent leakage.
- Produced of super soft material and designed for everyday use and comfort.
- Double Top Dry layer provides extra absorption at the front.



Application guide

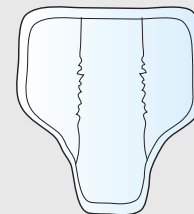


Unfold the pad. Do not touch the inside surface of the pad.

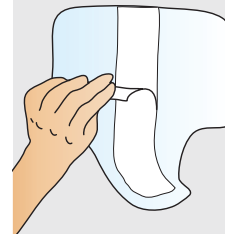

Remove the paper strip to expose the self-adhesive strip.



Place the pad in close-fitting underpants or fitting pants, broad side up to the front.

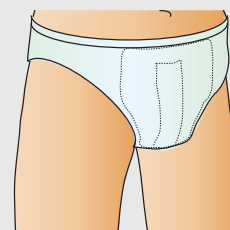

Put on the underpants.



Abri-Man Premium

Bed Protection

■ Abri-Man Zero, Premium



Art. no.	Product/ Colour Code	Size (mm)	Absorption Rothwell	Drops	Units per pack/ ctn
300740	■ Zero	90x180	200 ml	3	24/384

ABRI • MAN



Protective Wear

■ Abri-Man Formula, Premium



Art. no.	Product/ Colour Code	Size (mm)	Absorption Rothwell	Drops	Units per pack/ ctn
41006	■ Formula 1	290x230	450 ml	4	14/168
41007	■ Formula 2	290x230	700 ml	5	14/168

ABRI • MAN



Wound Care

■ Abri-Man Slippguard, Premium



Art. no.	Product/ Colour Code	Size (mm)	Absorption Rothwell	Drops	Units per pack/ ctn
207203	■ Slippguard	400x90	900 ml	6	20/100

ABRI • MAN



Body Wash

■ Abri-Man Special, Premium



Art. no.	Product/ Colour Code	Size (mm)	Absorption Rothwell	Drops	Units per pack/ ctn
300744	■ Special	730x370	2800 ml	9	21/84

ABRI • MAN



Skin Care

Gloves