

Shaped Pads

Abri-San Premium



Product features

- Breathable pad to maintain healthy skin and optimum comfort for the user.
- Top Dry-system, ensuring comfort and integrity of skin.
- Anatomically shaped pads to fit the user.
- Barriers to ensure the user protection against leakage
- Odour control for discretion.
- Abri-San Special available for fecal incontinence.
- Wetness indicator with graduation scale.



Application guide



Fold the pad length-wise with backsheet turned outwards. Pass pad from front to back. Position the upper rear end of pad at the coccyx.



Fan out the back of pad horizontally to create a channel shape between the legs.



Fan out pad horizontally at front.



Pull fitting pants up into position. If possible, the pant legs should be placed on the backsheet edges of the pad.

Abri-San Premium

■ Abri-San, Premium

Art. no.	Product/ Colour Code	Size (mm)	Absorption Rothwell	Drops	Units per pack/ ctn
9253	1	100x220	200 ml	2	28/280
9254	1A	100x280	200 ml	2	28/336
9260	2	100x260	350 ml	3	28/252
9266	3	110x330	500 ml	4	28/196
9267	3A	110x330	650 ml	4	28/196
9271	4	200x440	800 ml	5	28/168




■ Abri-San, Premium

Art. no.	Product/ Colour Code	Size (mm)	Absorption Rothwell	Drops	Units per pack/ ctn
9374	5	280x540	1200 ml	5	36/144
9378	6	300x630	1600 ml	6	34/102
9381	7	360x630	2100 ml	7	30/120
9382	8	360x630	2500 ml	8	21/84
9384	9	370x730	2400 ml	8	25/100
9386	10	370x730	2800 ml	9	21/84
9389	11	370x730	3400 ml	9	16/64
300200	Special	370x730	2000 ml	7	28/112

Abri-San Special is specially design for faecal incontinence. All Abri-San 5-11 should be used together with special fixation pants like Abri-Fix.




■ Abri-San, Premium, Bariatric range

Art. no.	Product/ Colour Code	Size (mm)	Absorption Rothwell	Drops	Units per pack/ ctn
300514	7/XXL	460x840	2000 ml	7	26/104
300515	11/XXL	460x840	3400 ml	9	14/56

Art. no. 300514 and 300515 are to be used with Abri-Fix Pants Super, size 4XL - 6XL. Art. no. 90697 - 90699.


