

3 Footrests and Seat Back




1

Align



2

Rotate

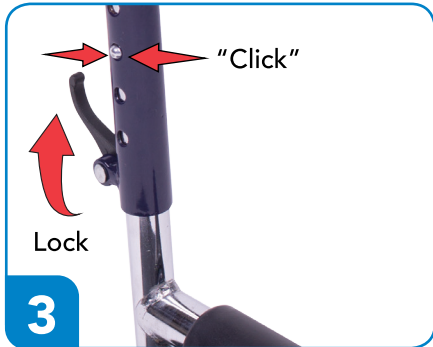

3

Adjust height as needed making sure buttons click into place and the lock is in the upright position.



4

Footrests should be lowered when in use.



5

Raise back handles until detents "click" into place.



Warning: Make sure detents buttons are securely locked into place.

TO PREVENT TIPPING WHILE SEATED, DO NOT PUSH YOURSELF WITH YOUR FEET.

Transport Chair

Fast & easy assembly, no tools required



1 Part Check

Transport chair frame



Footrests (2)



Front wheels (2)



Rear wheels (2)



Hand nuts (4)



Lock washers (4)



Hex bolts (2)



Note: actual items may vary from what is shown



Warning: To avoid serious injury, transport chair must be properly assembled. Ensure all bolts are fixed in place and hand nuts have been firmly tightened before each use.

2 Wheel Assembly




1 Insert rear wheel so it is on outside of frame, then add lock washer followed by hand nut to inside of frame.




2 Firmly tighten lock washer and hand nut.



3 Place front wheels into frame so the holes line up.

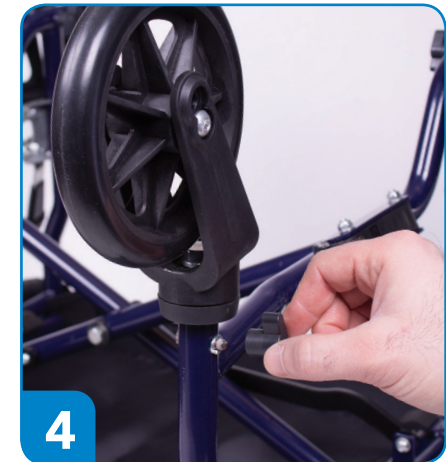

4 The interior punch out will hold the hex bolt in place as you firmly tighten the lock washer and hand nut.

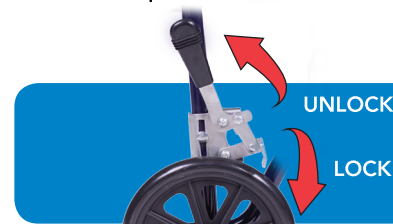

How to engage brakes: Lift up to unlock wheels and press down to lock