


10257

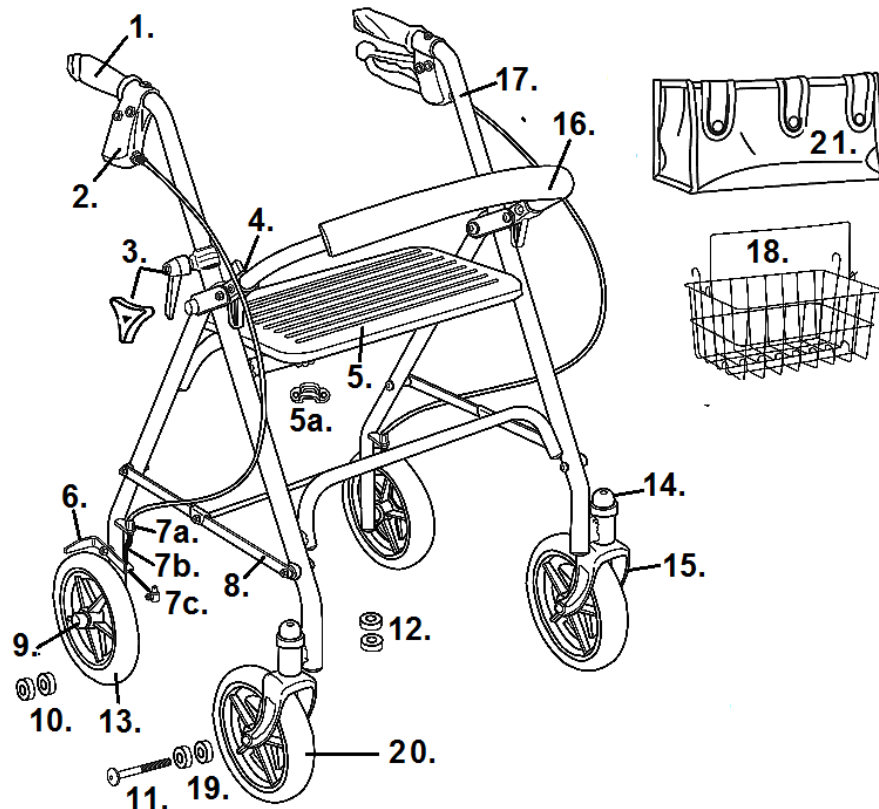

- |  |  |
|--|--|
| <p>1. Hand Grip Left- 95032F260N15<br/>Hand Grip Right- 95032F260N14</p> <p>2. Hand Brake &amp; Cable- 9502F1025701</p> <p>3. Height Adj. Ratchet (pr)- 95032F260N10<br/>Height Adj. Knob- 10208K [1]</p> <p>4. Backrest Bracket (pr)- 9501S72603</p> <p>5. Hard Seat &amp; Brackets- 9502F1025703<br/>Padded Seat &amp; Brackets- 9502F1025712</p> <p>5a. Hard Seat Brackets (pr)- 9502F1025702<br/>Padded Seat Brackets (pr)- 9502F1025713</p> <p>6. Wheel Lock Left- 9502F1025710<br/>Wheel Lock Right- 9502F1025711</p> <p>7a. Cable Adjuster- 9500001232</p> <p>7b. Cable Spring- 951021820000</p> <p>7c. Cable Fastener- 9500004101</p> <p>8. Folding Hinge Assembly- 9502F1025707</p> <p>9. Axle Cap- STDS001000</p> <p>10. Rear Wheel Bearing- STDS1011</p> <p>11. Caster Axle- 9502F1025709</p> | <p>12. Fork Stem Bearing- STDS1060</p> <p>13. Wheel Gray- 95032F260N09<br/>Wheel Black- 95032F260N09-BLK</p> <p>14. Dust Cap (pr)- 95032F260N06</p> <p>15. Fork Blue- 9502F1025708<br/>Fork Red- 9502F1025708R</p> <p>16. Backrest- 9502F1025705</p> <p>17. Handle Bar Only Left- 9502F1025715 [3]<br/>Handle Bar Only Right- 9502F1025714 [3]<br/>Handle Bar Assy. Left- 9503Z1025715 [1] [2]<br/>Handle Bar Assy. Right- 9503Z1025714 [1] [2]</p> <p>18. Basket- 9502F1025706</p> <p>19. Caster Bearing- STDS1011<br/>Caster Bearing- 1992M-19 [1]</p> <p>20. Caster Gray- 95032F260N09<br/>Caster Black- 95032F260N09-BLK<br/>Caster Black- 9503Z1025709 [1]</p> <p>21. Tote Bag- 9505W1026120</p> <p>Not Shown Tray- 809</p> |
|--|--|
- [1] 3Z serial number.  
[2] Includes parts 1 and 2. Handle height is 35"  
[3] Handle height is 37".