www.drivemedical.com

Rev.2.02.18.14

driveTM

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item # 12023

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

driveTM

installation instructions

1. Remove contents from box.
2. Attach red handle by threading plastic screws through the bottom of the bench and tightening.
3. Attach rubber grips by threading the plastic screws through the top of the bench and into the rubber grips, making sure the grey rubber side faces outward. This will be the side that makes contact with the bath tub.
Do not tighten.
4. Place the shower bench in the tub with the red handle on the outside. This handle provides support into and out of the tub.
5. Move rubber grips flush against the side of tub and tighten screws.
6. Make sure bath bench is secure before each use.

NOTE: to maintain to the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe shower bench dry after each use.

WEIGHT CAPACITY: 250 LBS (113 KG)

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

The logo consists of the word "drive" in a lowercase, bold, sans-serif font. The letter "d" is blue, the "r" is orange, the "i" is blue, the "v" is blue, and the "e" is blue. A small "TM" symbol is located at the top right of the "e".

© 2014 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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The logo for Drive Medical, featuring the word "drive" in a bold, lowercase, sans-serif font. The letter "i" has a small orange square above it. A trademark symbol (TM) is located at the top right corner of the letter "e".

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The logo for Drive Medical, featuring the word "drive" in a lowercase, bold, sans-serif font. The letter "i" has a yellow horizontal bar through it, and the letter "v" has a blue horizontal bar through it. A trademark symbol (TM) is located at the top right of the "e".

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.