
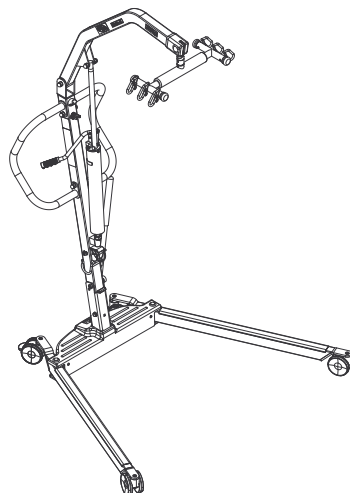
Invacare® I-Lift™

ILIFTHM, ILIFTEM, ILIFTEE

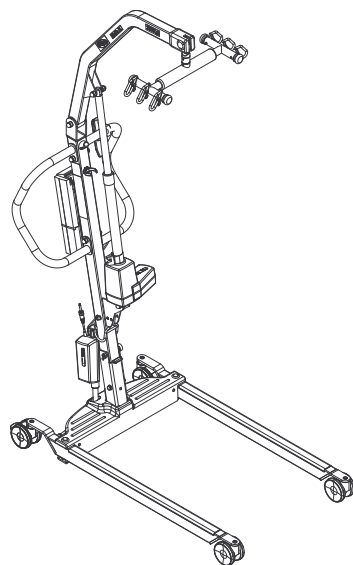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This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

© 2013 Invacare Corporation.

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

1	General	5
1.1	Symbols	5
2	Safety	6
2.1	General Guidelines	6
2.2	Operating Information	6
	General	6
	Pinch Points and Positioning	7
	Electrical and Grounding	7
	Disposal	8
2.3	Radio Frequency Interference	8
2.4	Product Labeling	9
3	Technical Data	10
3.1	Patient Lift	10
3.2	Full Body and Heavy Duty Slings	11
3.3	Divided Leg and Toileting Slings	12
3.4	I-Lift Scale ILS450	12
4	Assembly	13
4.1	Safe Assembly	13
4.2	Assembling the Mast to the Base	13
4.3	Assembling the Manual/Hydraulic Pump to the Boom	15
4.4	Assembling the Electric Actuator to the Boom	16
4.5	Installing the Leg Actuator to the Base	17
4.6	Installing the Shifter Handle	17
4.7	Checking the Service Light	18
5	Operation	19

5.1	Introduction	19
5.2	Closing/Opening Legs	19
	Closing/Opening Manual Legs	19
	Closing/Opening Electric Legs	20
5.3	Raising/Lowering the Lift	20
	Raising/Lowering a Manual/Hydraulic Lift	21
	Raising/Lowering an Electric Lift	21
5.4	Activating a Mechanical Emergency Release	22
	Primary Emergency Release	22
	Secondary Emergency Release	23
5.5	Performing an Emergency Stop	23
5.6	Mounting the Battery Charger	23
5.7	Charging the Battery	24
	Battery Indicator	24
	Using the Battery Charger to Charge the Battery	26
6	Lifting the Patient	27
6.1	Safe Lifting	27
6.2	Preparing to Lift	28
6.3	Attaching the Slings to the Lift	30
6.4	Lifting and Transferring the Patient	32
	Floor Transfer (Lifting from the Floor)	33
	Commode Transfer Guidelines	34
	Bed Transfer	34
	Wheelchair Transfer	35
7	Troubleshooting	36
7.1	Troubleshooting Table	36
8	Maintenance	38
8.1	Safe Maintenance	38
8.2	Maintenance Safety Inspection Checklist	38
8.3	Lubricating the Lift	40
8.4	Detecting Wear and Damage	40

8.5	Cleaning the Sling and Lift.	40
8.6	Resetting the Service Light	40
8.7	Maintaining the Manual/Hydraulic Pump.	41
8.8	Replacing a Manual/Hydraulic Pump.	41
8.9	Adjusting the Pump Control Valve	42
8.10	Replacing the Electric Actuator.	44
8.11	Replacing the Leg Actuator.	45
8.12	Checking and Tightening Mast Pivot Bolt	46
8.13	Replacing the Swivel Bar.	46
8.14	Maintaining the Base Adjustment.	47
8.15	Replacing Rear Casters.	48
8.16	Replacing Front Casters	48
9	Scale	49
9.1	Scale Introduction	49
9.2	Removing the Swivel Bar	49
9.3	Installing the Scale	50
9.4	Operating the Scale	51
9.5	Weighing the Patient	52
9.6	Replacing the Battery	53
9.7	Calibrating the Scale.	53
9.8	Troubleshooting	54
	Display Codes	54
10	Warranty.	55
10.1	Limited Warranty — North America	55
11	Survey	56
11.1	Usability Survey	56

I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

**WARNING!**

- Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION!**

- Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

**IMPORTANT**

- Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

Date of manufacture.

2 Safety

2.1 General Guidelines

**WARNING!**

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

The information contained in this document is subject to change without notice.

Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer for further instructions.

2.2 Operating Information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section.

For instance, for safety information related to assembling the lift, refer to 4.1 Safe Assembly, page 13.

General

**WARNING!****Risk of Falling**

The Invacare patient lift is NOT a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair). DO NOT attempt any transfer without approval of the patient's physician, nurse or medical assistant. Thoroughly read the instructions in this Owner's Manual, observe a trained team of experts perform the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient. Invacare slings and patient lift accessories are specifically designed to be used in conjunction with Invacare patient lifts. Slings and accessories designed by other manufacturers are not to be utilized as a component of Invacare's patient lift system.

- Use common sense in all lifts. Special care **MUST** BE taken with people with disabilities who cannot cooperate while being lifted.

Pinch Points and Positioning


WARNING!

Risk of Injury

Pinch points are present in several locations on the lift and fingers could be pinched.

The hanger bar can move suddenly and cause injury.

- ALWAYS keep hands and fingers clear of moving parts to avoid injury.
- When positioning lift, be aware of the position of the hanger bar and the patient. Injury could occur.

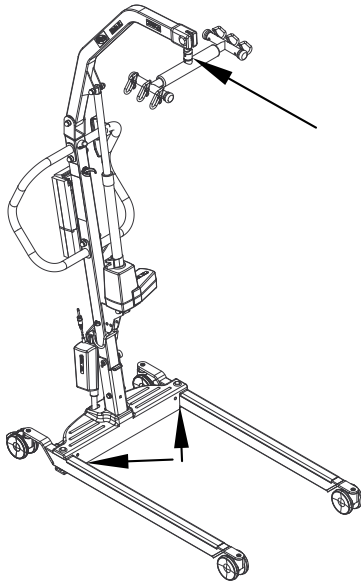Electrical and Grounding

WARNING!

GROUNDING INSTRUCTIONS

DO NOT, under any circumstances, cut or remove the round grounding prong from any plug used with or for Invacare products. Some devices are equipped with three-prong (grounding) plugs for protection against possible shock hazards. Where a two-prong wall receptacle is encountered, it is the personal responsibility and obligation of the customer to contact a qualified electrician and have the two-prong receptacle replaced with a properly grounded three-prong wall receptacle in accordance with the National Electrical Code. If you must use an extension cord, use ONLY a three-wire extension cord having the same or higher electrical rating as the device being connected. In addition, Invacare has placed RED/ORANGE warning tags on some equipment. DO NOT remove these tags.

- Carefully read battery/battery charger information prior to installing, servicing or operating your patient lift.
- To reduce the risk of electric shock, this product has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. DO NOT change the plug in any way.

Disposal

WARNING!

Environmental Hazard

This product has been supplied from an environmentally aware manufacturer that complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2002/96/CE.

Device contains lead acid batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste. They **MUST** be taken to a proper disposal site. Contact your local waste management company for information.
- Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

2.3 Radio Frequency Interference

WARNING!

- Most electronic equipment is influenced by Radio Frequency Interference (RFI). **CAUTION** should be exercised with regard to the use of portable communication equipment in the area around such equipment. If RFI causes erratic behavior, **PUSH** the Red Power Switch **OFF IMMEDIATELY**. **DO NOT** turn the Power Switch **ON** while transmission is in progress.

2.4 Product Labeling

Electric lift shown. Locations are the same on the hydraulic/manual lift.

WARNING

BEFORE using the Patient Lift, READ and UNDERSTAND the Owner's Manual for proper operation and safety procedures.

WARNING

WEIGHT LIMITATION 450 lbs (205 kg).

The Invacare Patient lift is NOT a transport device.

DO NOT roll casterbase over uneven surfaces that may cause the Patient Lift to tip over.

DO NOT lock the casters of the Patient Lift when lifting an individual. Casters **MUST** be left unlocked to allow Patient Lift to stabilize during lifting procedures.

USE ONLY Invacare slings and lift accessories.

For maintenance and replacement, use only components designed for this patient lift model. See the product label for model information.

Customer Service
1-800-333-6900

1073698 Rev D

Sling Sizes

Small
Navy

Medium
Purple

Large
Green

XL
Blue

XXL
Black

WARNING

USE ONLY INVACARE SLINGS and LIFT ACCESSORIES.

1154553 REV. A

3 Technical Data

3.1 Patient Lift

	Hydraulic — Manual Base	Electric — Manual Base	Electric — Electric Base
	ILIFTHM	ILIFTEM	ILIFTEE
Height at Sling Hook-up - MAX.:	71.5 in / 1816 mm	76.1 in / 1933 mm	
Height at Sling Hook-up - MIN.:	22.8 in / 579 mm	22.2 in / 564 mm	
Base Width OPEN:	47 in / 1196 mm		53.4 in / 1358 mm
Base Width CLOSED:	27.8 in / 705 mm		
Base Height (Clearance):	4.6 in / 117 mm		
Base Length:	46.6 in / 1184 mm		
Caster Size:	3.94 in / 100 mm		
Sling Options:	3 Styles	3 Styles	1 Style
Turning Diameter	57 in / 1450 mm	57.6 in / 1464	
Max Lifting Capacity (Weight Capacity):	450 lb / 205 kg		
Total Weight	110 lb / 49.9 kg	117.5 lb / 53.3 kg	120 lb / 54.4 kg
Cycles from a Full Charge (Varies depending upon load and stroke):	N/A	Approximately 40 cycles	
Battery:	N/A	24V DC (Rechargeable)	
Charger Input:	N/A	100-240V AC	
Charger Output/Charging Time:	N/A	29.5V DC Max 6 hrs	
Accessories:	Digital Scale		
Audio Low Battery Alarm:	N/A	Yes	
Motor Safety Devices:	N/A	Anti-Entrapment	
Warranty Pump/ Electronics:	1 Year		
Warranty Frame	3 years		

3.2 Full Body and Heavy Duty Slings

	Full Body			Full Body w/Commode			Heavy Duty w/o Commode	Heavy Duty w/ Commode
	R110 R112*	R111 R113*	R114	R115	R116	R117*	R140*	R141*
Size:	M	L	XL	M	L	XL	N/A	N/A
Width:	41.5	45.5	45.5	41.5	45.4	45.5	45	45
Commode Opening:	N/A	N/A	N/A	8	8	9	N/A	7
Length:	54.7	60.5	65.3	54.7	60.5	65.3	55	55
Commode Opening:	N/A	N/A	N/A	11	11	13	N/A	13
Back:	N/A	N/A	N/A	N/A	N/A	N/A	N/A	N/A
Weight Capacity (lbs):	450	450	450	450	450	450	600	600

*Slings made of a mesh material.

All dimensions are in inches except where noted.

3.3 Divided Leg and Toileting Slings

	Divided Leg				Toileting
	R100P	R100	R101	R102	R121
Size:	P	M	L	XL	L
Width:	33	37.5	41	44.5	36
Commode Opening:	N/A	N/A	N/A	N/A	N/A
Length:	59.8	62.8	67.8	72.3	37
Commode Opening:	N/A	N/A	N/A	N/A	N/A
Back:	29.2	35.7	41.7	43.2	N/A
Weight Capacity (lbs):	450	450	450	450	450

All dimensions are in inches except where noted.

3.4 I-Lift Scale ILS450

WARNING!

- The weight capacity is limited to the lowest rated capacity of any one of the components in use (e.g. Patient Lift, Sling or Scale). The patient's weight **MUST** not exceed the lowest rated capacity of any component.

Weight Range:	Up to 600 lbs. (272.7 Kg)
Resolution:	+/- 0.2 lbs (.1 Kg)
Display:	Liquid Crystal Digital
Automatic Power Down:	Two (2) minutes
Size:	5-3/4 L X 3-5/8 W X 4 H
Weight:	.55 lbs
Power:	Nine (9) Volt Alkaline battery (included)
Battery Life:	Approximately 1500 readings
Temperature Range:	50° To 104°F (10° To 40°C)

4 Assembly

4.1 Safe Assembly

**WARNING!****Risk of Injury**

Improper assembly may cause injury or damage.

- Use only Invacare parts in the assembly of this patient lift. The base legs, the mast, boom, pump assembly and the hanger bar are manufactured to specifications that assure correct alignment of all parts for safe functional operation.
- DO NOT overtighten the mounting hardware. This will damage the mounting bracket.


4.2 Assembling the Mast to the Base

1. Lock rear casters Ⓐ (Detail A).
2. Remove the shoulder bolt Ⓑ, nut Ⓒ and washers Ⓓ, from the mast Ⓔ.
3. Raise the mast assembly to an upright position until it locks in place (Detail B).
4. Install the shoulder bolt and washers, through the mast and the base Ⓕ.
5. Secure with the nut.


**WARNING!**

- The mast may be folded for storage or transporting. Each time the mast is folded, the mast **MUST** be properly secured to the base assembly.


A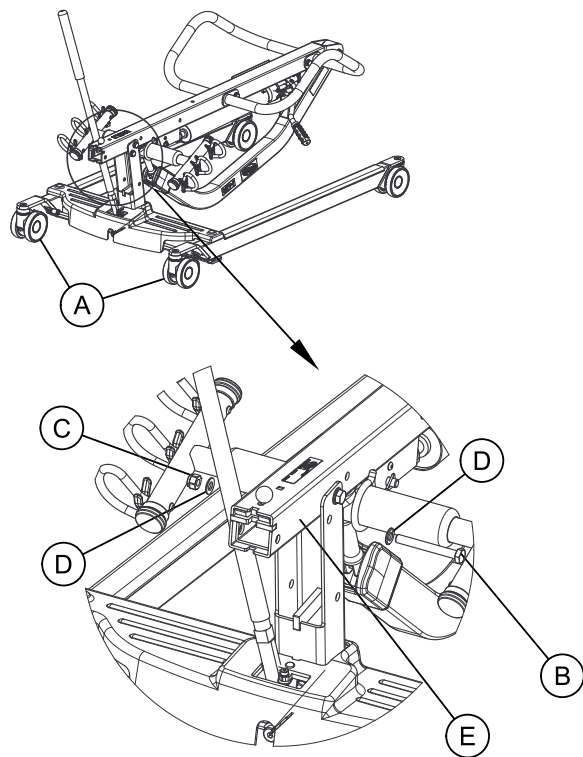

B

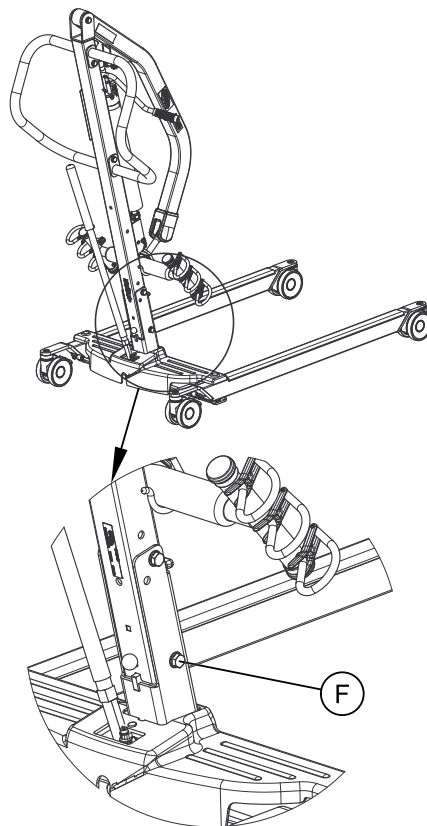4.3 Assembling the Manual/Hydraulic Pump to the Boom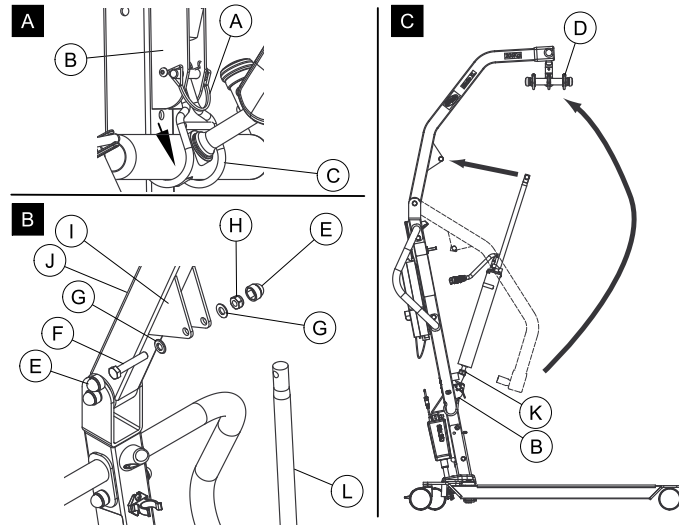

 The bottom of the hydraulic pump assembly (K) will already be assembled to the mast mounting bracket (Detail C).

4. Lift-up on the boom and place it on your left shoulder.
5. Let the hydraulic pump rest on the right-side of your chest and rotate the shaft extension (L) of the hydraulic pump assembly until it lines-up with the mounting bracket holes on the boom.
6. Align the holes of the mounting bracket with those of the shaft extension
7. Install the bolt, washers, nut and caps.

 Be sure that the bolt is completely through the holes of the boom mounting bracket and the manual/hydraulic pump assembly. The boom assembly will pivot easily if the mounting hardware is aligned properly when the boom is secured to the mast.


CAUTION!

– DO NOT overtighten the mounting hardware. This will damage the mounting bracket.

1. Remove the quick release pin (A) from the mast mounting bracket (B) (Detail A).
2. Lift the hook (C) securing the hanger bar (D) to the mast.
 The hanger bar is not shown in the hook in Detail A.
3. Remove the caps (E), bolt (F), washers (G) and nut (H) from the mounting bracket (I) on the boom (J) (Detail B).

4.4 Assembling the Electric Actuator to the Boom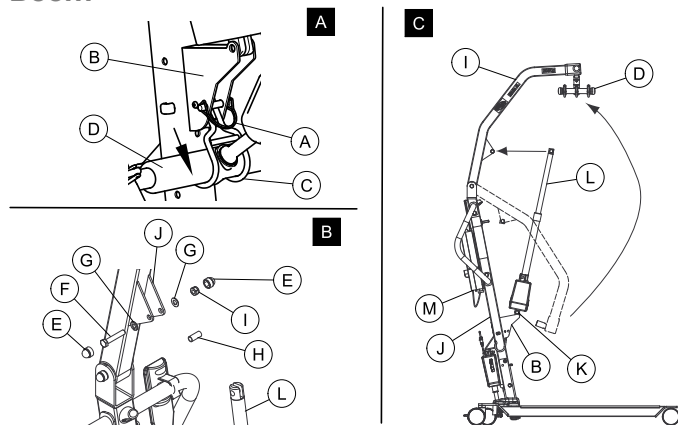

CAUTION! Risk of Damage

– DO NOT overtighten the mounting hardware. This will damage the mounting bracket.

1. Remove the quick release pin (A) from the mast mounting bracket (B) (Detail A).
2. Lift the hook (C) securing the hanger bar (D) to the mast.
3. Remove the caps (E), bolt (F), washers (G), bushing (H) and nut (I) from the boom mounting bracket (J) (Detail B).


 The bottom of the electric actuator assembly (K) will already be assembled to the mast mounting bracket (Detail C).

4. Lift-up on the boom and place it on your left shoulder.

5. Let the actuator rest on your right-side of your chest and rotate the shaft extension (L) of the actuator assembly until it lines-up with the mounting holes in the boom assembly.
6. Align the holes of the boom assembly mounting bracket with those of the actuator


 Be sure that the bolt is completely through the holes of the boom mounting bracket and the actuator assembly. The boom assembly will pivot easily if the mounting hardware is aligned properly when the boom is secured to the mast.
7. Install the bolt, washers, nut, bushing and caps.
8. Plug in the electrical connectors (not shown) from the electric actuator to the bottom of the battery assembly.
9. Plug the pendant control into the bottom of the battery assembly (M). Be sure the pendant control plug is fully inserted.

4.5 Installing the Leg Actuator to the Base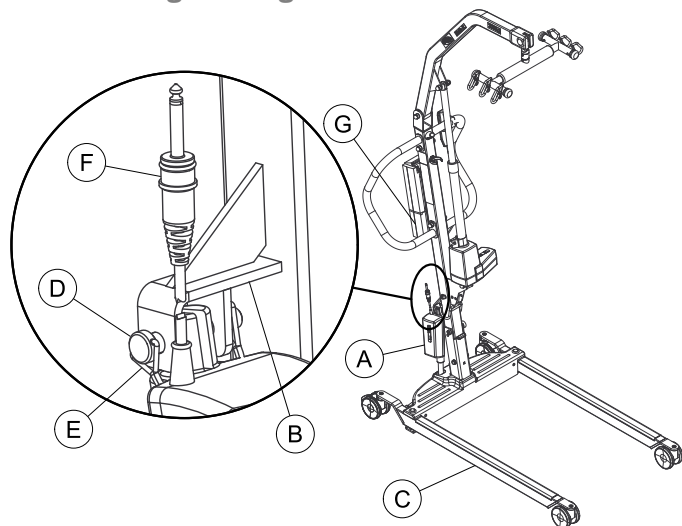

1. Perform the following to secure the leg actuator to the mast bracket:
 - a. Position leg actuator (A) into the mast bracket (B).
 - b. Move the legs (C) to align the holes in the leg actuator with the holes in the mast bracket.
 - c. Install the pin (D) through the holes of the leg actuator and mast bracket and secure with hitch pins (E).
2. Connect the leg actuator cable (F) to the bottom of the control box (G).
3. Ensure the actuator plug is fully pushed into the control box.
4. Check the service light. Refer to 4.7 Checking the Service Light, page 18.


4.6 Installing the Shifter Handle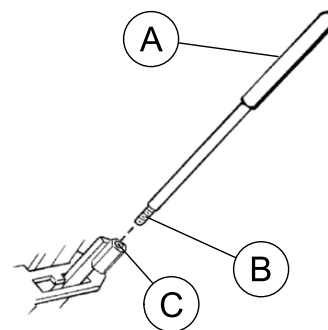

i This procedure applies to the ILIFTHM and ILIFTEM only.

1. Remove the shifter handle (A) from the packaging carton.
2. Line-up the shifter handle threads (B) with threaded opening (C) in the base.
3. Turn the shifter handle clockwise and securely tighten into the base.

4.7 Checking the Service Light

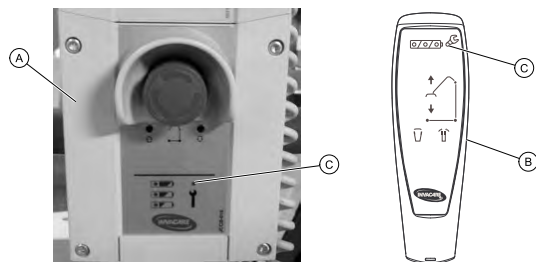This procedure applies to the ILIFTEE models only.

Each time the lift is assembled, and before using the lift, the service light should be checked.

1. Assemble the lift.
2. Examine the control box (A) or the hand pendant (B) to see if the service light (C) is flashing.
3. Refer to the table.

Service Light	Initial Assembly	Reassembly
Flashing	Reset the service light. Refer to 8.6 Resetting the Service Light, page 40.	The lift requires service. Contact your local Invacare dealer for service.
Not Flashing	The lift is ready for use.	

5 Operation

5.1 Introduction

The operation of the patient lift is an easy and safe procedure.

 Before using the lift with a patient, refer to the following procedures for safety information and instruction:

- 2.2 Operating Information, page 6
- 6.4 Lifting and Transferring the Patient, page 32

5.2 Closing/Opening Legs

WARNING!

Risk of Injury


The lift could tip and endanger the patient and assistants.

- The legs of the lift must be in the maximum open position for optimum stability and safety. If it is necessary to close the legs of the lift to maneuver the lift under a bed, close the legs of the lift only as long as it takes to position the lift over the patient and lift the patient off the surface of the bed. When the legs of the lift are no longer under the bed, return the legs of the lift to the maximum open position.
- The shifter handle must be locked in place for optimum stability and safety.


Closing/Opening Manual Legs

The shifter handle is used to open or close the legs of the base for stability when lifting a patient.

A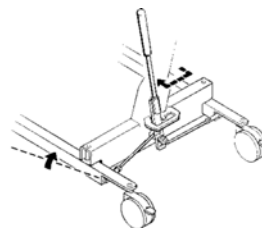

B

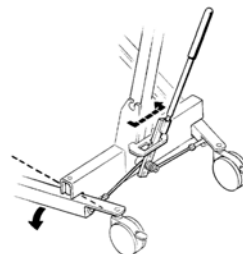 Refer to the safety information in 5.2 Closing/Opening Legs, page 19 before performing this procedure.

- I. Stand at the rear of the patient lift and grasp the shifter handle with one hand and place the opposite hand on the steering handle of the mast for balance.
-  The shifter handle **MUST** lock into its mounting slot to lock the legs in the full closed position.

**WARNING!****Risk of Injury**

The lift could tip and endanger the patient and assistants.
 – If the shifter handle is NOT positioned completely into its mounting slot, DO NOT use the patient lift until the shifter handle is properly seated and the legs of the patient lift are locked in place.

2. Perform one of the following:


- a. Closing the Legs Ⓐ — Pull the shifter handle away from the patient lift and then to your left until it locks in the notch of the bracket.
- b. Opening the Legs Ⓑ — Pull the shifter handle away from the patient lift and then to your right.

Closing/Opening Electric Legs

The pendant is used to open or close the legs of the base for stability when lifting a patient.

Refer to the safety information in 5.2 Closing/Opening Legs, page 19 before performing this procedure.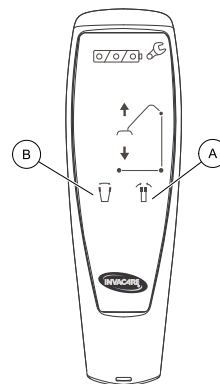

1. To close the legs, press and hold the legs closed button Ⓐ.
2. To open the legs, press and hold the legs open button Ⓑ.



The legs will stop moving when the button is released.

5.3 Raising/Lowering the Lift**WARNING!****Risk of Injury**

The lift could tip and endanger the patient and assistants.
 – Invacare does not recommend locking of the rear casters of the patient lift when lifting an individual.
 – Invacare does recommend that the rear casters be left unlocked during lifting procedures to allow the patient lift to stabilize itself when the patient is initially lifted from a chair, bed or any stationary object.

Raising/Lowering a Manual/Hydraulic Lift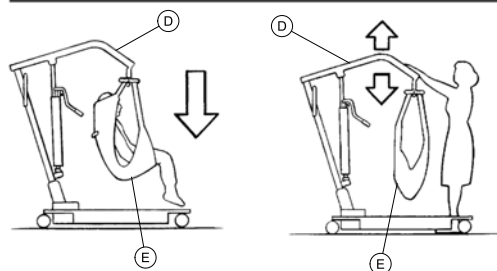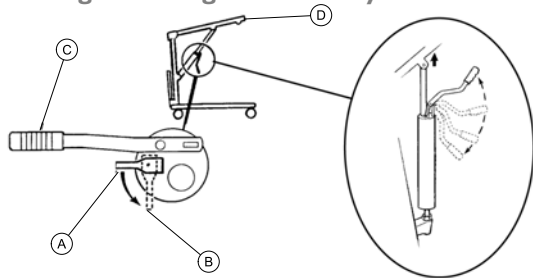

 Refer to the safety information in 5.3 Raising/Lowering the Lift, page 20 before performing this procedure.

There are two controls on the pump assembly:

- The control valve
- The pump handle

1. To raise the lift:

- a. Position the control valve in the closed position **A** (towards the pump handle **C**).

- b. Move the pump handle up and down to elevate the boom **D** until it locks in the notch of the bracket.
- c. Ensure the boom has locked in the notch of the bracket.

2. To lower the lift:


- a. Gently open **B** the control valve (away from the pump handle).
- b. If there is no patient in the sling **E**, pull down on the boom.

 The rate of descent of the boom is controlled by the amount that the control valve is open.

 A safety gate is part of the hydraulic system that controls the maximum descent of the boom regardless of how far the control valve is opened.

Raising/Lowering an Electric Lift

 Refer to the safety information in 5.3 Raising/Lowering the Lift, page 20 before performing this procedure.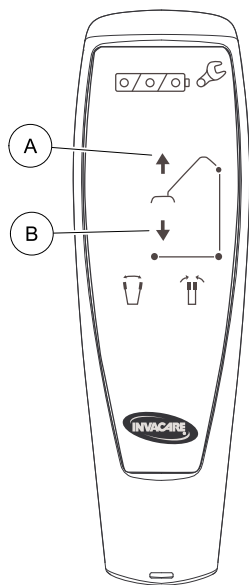

1. To raise the lift — Press and hold the UP **A** button to raise the boom and the patient.
2. To lower the lift — Press and hold the DOWN **B** button to lower the boom and the patient.


Release the button to stop raising or lowering the lift.

5.4 Activating a Mechanical Emergency Release


There are two types of mechanical emergency release — primary and secondary.

Primary Emergency Release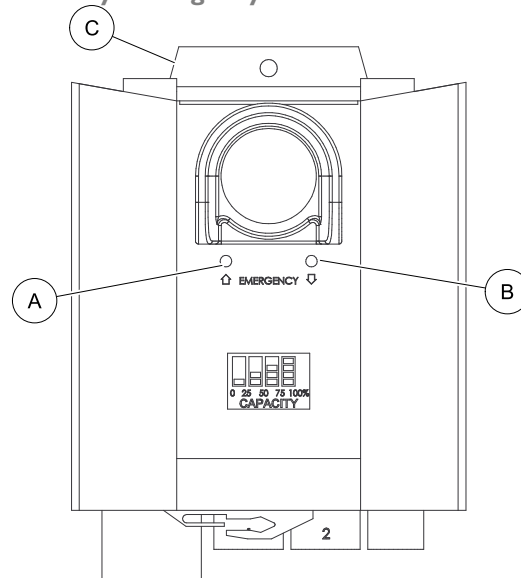

1. Insert a pen into the hole labeled Emergency Up **A** or Emergency Down **B** on the control box **C**.

Secondary Emergency Release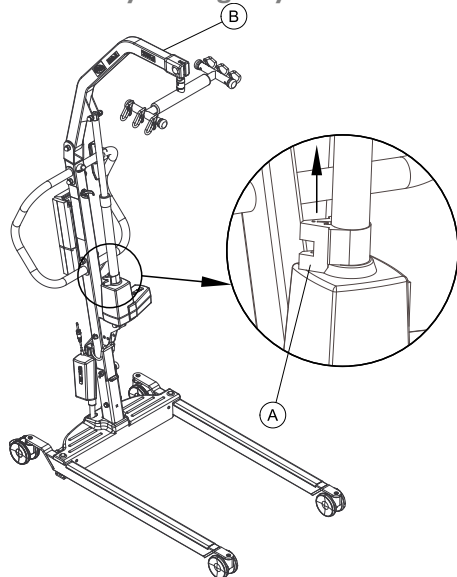


i It is recommended that the primary emergency release be used. The secondary emergency release is only a back-up to the primary emergency release.

In cases where the primary release is either not functioning or unreachable, a secondary emergency release may be used.


1. Pull up on the EMERGENCY grip (A) and push down on the boom (B) at the same time.

5.5 Performing an Emergency Stop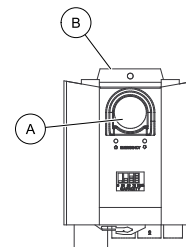

1. Press the RED button (A) on the control box (B) in to stop the boom and patient from raising or lowering.
2. To reset, rotate the emergency button clockwise.

5.6 Mounting the Battery Charger

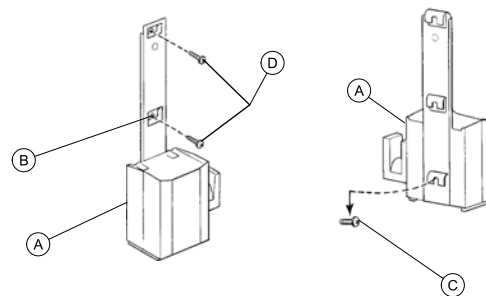i Refer to your local regulations concerning proper mounting procedures.

1. Place the battery charger with mounting bracket (A) on the wall at the desired position.
2. With a pencil, mark the middle hole (B) position.
3. Measure down 6½ inches from the pencil mark and drill one mounting hole.
4. Install the bottom mounting screw (C) until there is an approximate 1/8-inch gap between the screw head and the wall.
5. Install the battery charger with mounting bracket onto the bottom mounting screw.
6. Drill the remaining two mounting holes.
7. Install the two remaining mounting screws (D) through the mounting bracket and into the wall. Tighten securely.
8. Plug the battery charger into the wall electrical outlet.



ON LED should illuminate.

5.7 Charging the Battery

Invacare recommends the battery be recharged daily to prolong battery life.


Battery Indicator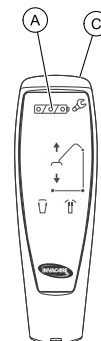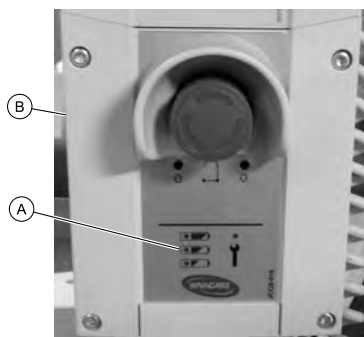

The battery indicator (A) is located on the control box (B) and on the hand pendant (C). The LEDs indicate the battery state:

Hand Pendant Battery Indicator 	Control Box Battery Indicator 	Battery State	Description
		Full Charge	The battery is OK — no need for charging (100–50%). The third LED is GREEN.
		Partial Charge	The battery needs to be charged (50–25%). The second LED is YELLOW.
		Low Charge	The battery needs to be charged (Less than 25%). The horn beeps when a button is pressed. The first LED is YELLOW.
		Low Charge (LED blinking)	The battery needs to be charged. Some of the functionality of the lift is lost and it is only possible to lower the boom. An audible alarm will sound (horn will beep) when battery is low. If the audible alarm sounds during a transfer, complete that transfer and then charge the battery.

Using the Battery Charger to Charge the Battery

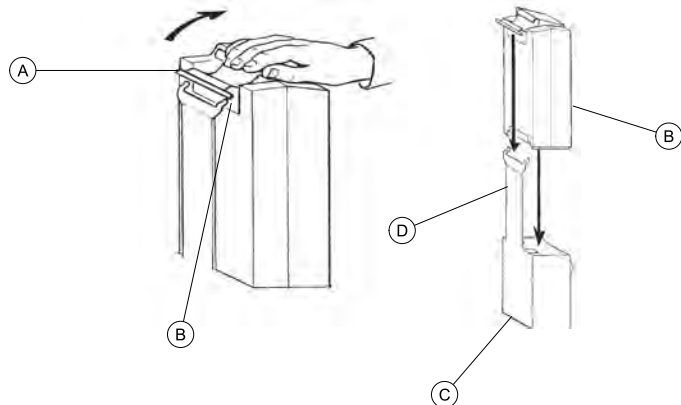1. Lift up on the handle (A) on the back of the battery (B).
2. Lift the battery up and out away from the control box (C).

CAUTION!

Mounting the battery improperly may cause injury or damage.

- Make sure there is an audible click when mounting battery on the battery charger to confirm proper mounting.

3. Place the battery on the battery charger (D) as shown. Make sure there is an audible click.

The charge LED will illuminate. When charging is complete, charge LED will stop illuminating.

A battery needing to be fully recharged will take approximately four hours.

4. Lift up on the handle on the back of the battery.

5. Lift the battery up and out away from the battery charger.

CAUTION!

Mounting the battery improperly may cause injury or damage.

- Make sure there is an audible click when mounting battery on the control box to confirm proper mounting.

6. Reinstall the battery onto the control box as shown. Make sure there is an audible click.

The battery mounts to the control box and battery charger as shown.

6 Lifting the Patient

6.1 Safe Lifting

WARNING!

Risk of Injury

The lift could tip and endanger the patient and assistants.

- Refer to the safety information and instructions in the following procedures **BEFORE** performing this procedure:

6.2 Preparing to Lift, page 28

6.3 Attaching the Slings to the Lift, page 30

6.4 Lifting and Transferring the Patient, page 32

- Although Invacare recommends that two assistants be used for all lifting preparation, transferring from and transferring to procedures, our equipment will permit proper operation by one assistant. The use of one assistant is based on the evaluation of the health care professional for each individual case.
- **DO NOT** exceed maximum weight limitation of the patient lift. The weight limitation for the I-Lift is 450 lbs (205 kg).
- **DO NOT** attempt any transfer without approval of the patient's physician, nurse or medical assistant. Thoroughly read the instructions in this Owner's Manual, observe a trained team of experts perform the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient.

WARNING!

Risk of injury

The lift could tip and endanger the patient and assistants.

- During transfer, with patient suspended in a sling attached to the lift, **DO NOT** roll caster base over uneven surfaces that could cause the patient lift to tip over.
- Use steering handle on the mast at all times to push or pull the patient lift.
- Wheelchair wheel locks **MUST** be in a locked position before lowering the patient into the wheelchair for transport.
- Before transferring, check that the wheelchair weight capacity can withstand the patient's weight.

6.2 Preparing to Lift

Refer to the Safety section in this manual and review the information in 6.1 Safe Lifting, page 27 before proceeding further and observe all warnings indicated.

Before positioning the legs of the patient lift under a bed, make sure that the area is clear of any obstructions.


WARNING!


Risk of Injury

The lift could tip and endanger the patient and assistants.


- The legs of the lift must be in the maximum open position for optimum stability and safety. If it is necessary to close the legs of the lift to maneuver the lift under a bed, close the legs of the lift only as long as it takes to position the lift over the patient and lift the patient off the surface of the bed. When the legs of the lift are no longer under the bed, return the legs of the lift to the maximum open position.
- The shifter handle must be locked in place for optimum stability and safety.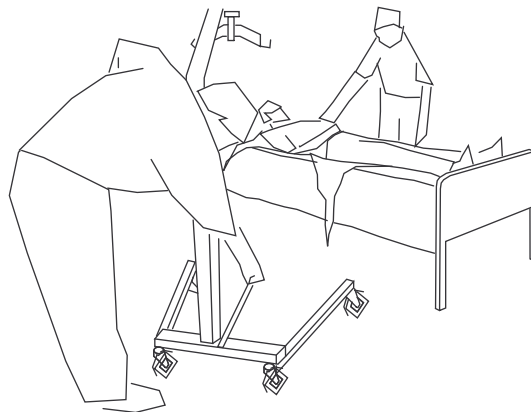

1. Position the patient onto the sling. Refer to your sling owner's manual.
2. Unlock the rear casters.
3. Open and lock the legs. Refer to 5.2 Closing/Opening Legs, page 19.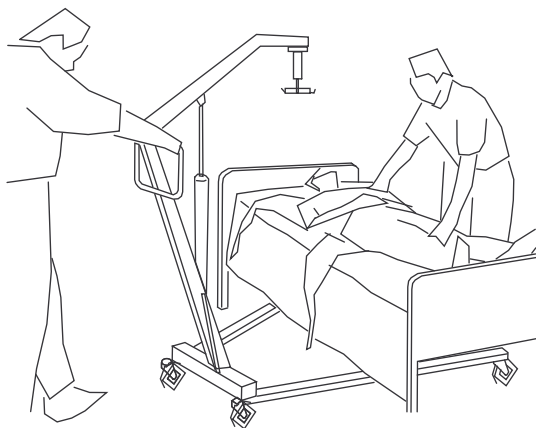

4. Use the steering handle to push the patient lift into position.

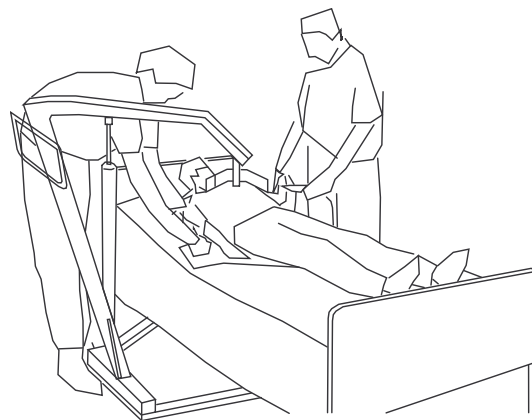5. Lower the patient lift for easy attachment of the sling.
6. Lock the rear casters.
7. Proceed to 6.4 Lifting and Transferring the Patient, page 32.

6.3 Attaching the Slings to the Lift

WARNING!

Risk of Falling

Improperly installed slings or damaged slings can cause the patient to fall or cause injury to assistants.

- Use an Invacare approved sling that is recommended by the individual's doctor, nurse or medical assistant for the comfort and safety of the individual being lifted.
- After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tears, and loose stitching.
- Bleached, torn, cut, frayed, or broken slings are unsafe and could result in injury. Discard immediately.
- DO NOT alter slings.
- Be sure to check the sling attachments each time the sling is removed and replaced, to ensure that it is properly attached before the patient is removed from a stationary object (bed, chair or commode).

WARNING!

- If the patient is in a wheelchair, secure the wheel locks in place to prevent the chair from moving forwards or backwards.
- DO NOT remove the strap retainers from the hanger bar hooks. The strap retainers must be closed when the straps are properly installed on the hooks.
- When connecting slings equipped with color coded straps to the patient lift, the shortest of the straps MUST be at the back of patient for support. Using long section will leave little or no support for patient's back. The loops of the sling are color coded and can be used to place patient in various positions. The colors make it easy to connect both sides of the sling equally. Make sure that there is sufficient head support when lifting a patient.

The slings have color coded straps to assist with proper attachment.

1. Place the straps (A) of the sling (B) over hooks (C) of the hanger bar (D).




 The strap retainer (F) closes when the sling strap is properly seated on the hook (C).

2. Match the corresponding colors on each side of the sling for an even lift of the patient.

 Model Nos. R110 - R117 Full Body Slings and Model Nos. R120 - R122 Toileting Slings have four sling straps (two per side — See Detail A). Model Nos. R100 - R102 Divided Leg Slings have six sling straps (three per side — See Detail B).

Invacare Lift Swivel Bars have three hookup points per side. The middle hookup (E) is **ONLY** used for slings that have three sets of straps per side.

3. Use the lift. Refer to 6.4 Lifting and Transferring the Patient, page 32.

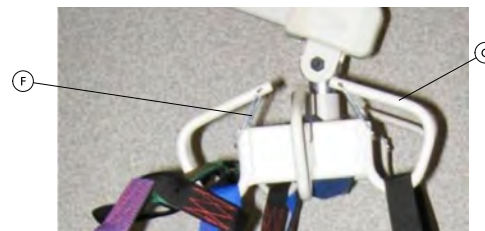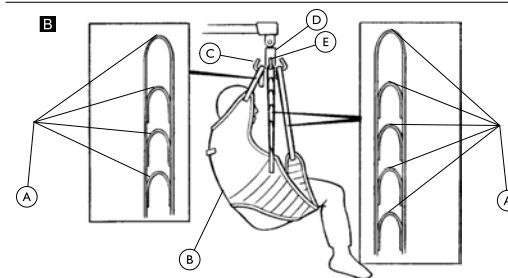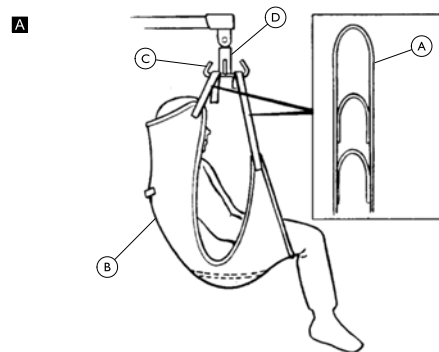6.4 Lifting and Transferring the Patient

WARNING!

Risk of Injury

The lift could tip and endanger the patient and assistants.

- Refer to the safety information and instructions in the following procedures **BEFORE** performing this procedure:


6.1 Safe Lifting, page 27

6.2 Preparing to Lift, page 28

6.3 Attaching the Slings to the Lift, page 30

5.3 Raising/Lowering the Lift, page 20

- Invacare does not recommend locking of the rear casters of the patient lift when lifting an individual.
- Invacare recommends locking the rear hanger casters **ONLY** when positioning or removing the sling from around the patient.
- Invacare does recommend that the rear casters be left unlocked during lifting procedures to allow the patient lift to stabilize itself when the patient is initially lifted from a chair, bed or any stationary object.

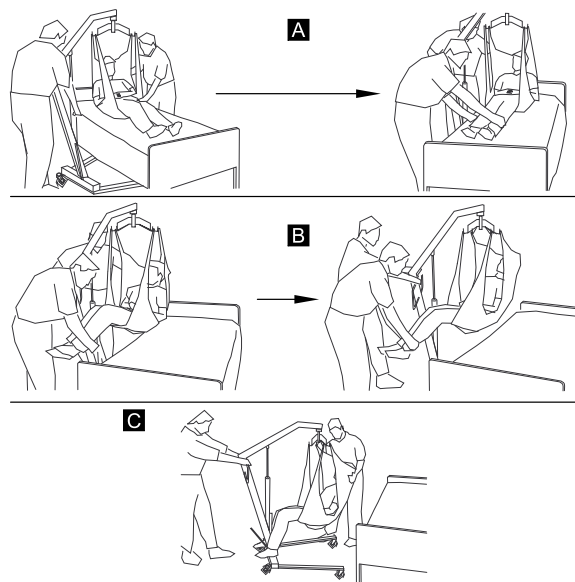1. Move the lift to the patient area and prepare to lift. Refer to 6.2 Preparing to Lift, page 28.
2. Attach the sling to the lift. Refer to 6.3 Attaching the Slings to the Lift, page 30.
3. Lift the patient high enough to clear the stationary object with their weight fully supported by the lift. Refer to 5.3 Raising/Lowering the Lift, page 20.

On manual/hydraulic lift, the boom will stay in position until the control valve is opened. On the electric lift, the boom will stay in position until the **DOWN** (▼) button is pressed.

**WARNING!****Risk of Injury**

An improperly attached sling could cause the patient to fall.

An improperly adjusted sling could cause injury to the patient.

- Adjustments for safety and comfort should be made before moving the patient.
- Patient's arms should be inside of the straps.

4. Before moving the patient, check again to make sure that the sling is properly connected to the hooks of the hanger bar. If any attachments are not properly in place, lower the patient back onto the stationary object and correct this problem.
5. Using the steering handle, move the lift away from the stationary object.
6. Using the handles on the sling, turn the patient so that he/she faces the assistant operating the patient lift (Detail "C").
7. Lower the patient so that his feet rest on the base of the lift, straddling the mast.

The lower center of gravity provides stability making the patient feel more secure and the lift easier to move.

8. Move the patient lift with both hands firmly on the steering handle.

Read and understand the information that pertains to transfer to or from specific types of surfaces **BEFORE** performing this procedure:

- Bed Transfer, page 34
- Floor Transfer (Lifting from the Floor), page 33
- Wheelchair Transfer, page 35
- Commode Transfer Guidelines, page 34

9. Raise or lower the lift to position the patient over the stationary surface.

Be sure to raise or lower the patient enough to clear the sides of the stationary object.

10. Lower the patient onto the stationary surface.
11. Lock the rear casters.
12. Detach the sling from the hanger bar.

The sling can be left under the patient, if desired.

13. Unlock the rear casters.
14. Move the lift away from the area.

Floor Transfer (Lifting from the Floor)

Perform these steps in addition to those in 6.4 Lifting and Transferring the Patient, page 32 when transferring from the floor:

1. Determine if the patient has suffered any injuries from a fall. If no medical attention is needed, proceed with the transfer.
2. Position the sling under the patient.

Refer to the Patient Slings owner's manual, part number 1023891, for more information about positioning slings.

3. One assistant should have the patient bend his knees and raise his head off of the floor.

This assistant should support the patient's head with a pillow.

4. The other assistant should open the legs of the lift.
5. Position the lift with one leg under the patient's head and the other leg under the patient's bent knees.

Keep the sling straps inside of the legs of the lift.

6. Lower the boom so the hanger bar is directly over the patient's chest.
7. Attach the sling and proceed with the transfer. Refer to 6.4 Lifting and Transferring the Patient, page 32.

Ensure the patient's arms are inside of the sling.

Commode Transfer Guidelines

Perform these steps in addition to those in 6.4 Lifting and Transferring the Patient, page 32 when transferring to or from a commode.

The slings with commode openings are designed to be used with either a commode chair or standard commode.

1. Before transferring the patient, the patient lift should be guided to the bathroom facilities to check that it can be easily maneuvered towards the commode.
 - 

The Invacare patient lift is NOT intended as a transport device. If the bathroom facilities are NOT near the bed or if the patient lift cannot be easily maneuvered towards the commode, then the patient MUST be transferred to a wheelchair and transported to the bathroom facilities before using the patient lift again to position the patient on a standard commode.
2. Attach the slings to the lift. Refer to 6.3 Attaching the Slings to the Lift, page 30.
3. Elevate the patient high enough to clear the commode chair arms and have their weight supported by the patient lift. Refer to 5.3 Raising/Lowering the Lift, page 20
4. Both assistants should help guide the patient onto the commode.
5. Lower the patient onto the commode, leaving the sling attached to the hanger bar hooks.

Invacare recommends that the sling remain connected to the hanger bar hooks during the patient's use of either the commode chair or standard commode.



6. When complete, recheck for correct sling attachment.
7. Raise the patient off of the commode.
8. When the patient is clear of the commode surface, use the steering handles to move the lift away from the commode.
9. Perform one of the following:
 - Return the patient to the bed. Reverse the procedures in:
 - 6.4 Lifting and Transferring the Patient, page 32
 - 5.3 Raising/Lowering the Lift, page 20
 - 6.3 Attaching the Slings to the Lift, page 30
 - Return the patient to a wheelchair. Refer to Wheelchair Transfer, page 35.

Bed Transfer

Use the following guidelines when transferring to or from a bed:

- Position the patient as far over the bed as possible.
- If patient is being transferred from a surface that is lower than the bed, press the up arrow button to raise the patient above the surface of the bed. The patient should be elevated just high enough to clear the bed with their weight fully supported by the lift.
- When the patient is clear of the bed surface, swing their feet off the bed (Detail "B").
- After transfer, unhook the sling from all attachment points on the lift and remove the sling from around the patient.

Wheelchair Transfer

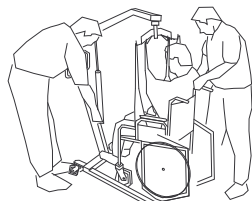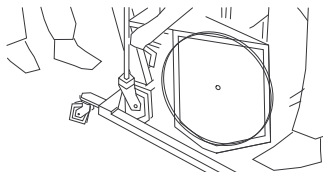WARNING!

Risk of Injury

- Before transferring, check that the wheelchair weight capacity can withstand the patient's weight.
- The wheelchair wheel locks **MUST** be in a locked position before lowering the patient into the wheelchair for transport.

Perform these steps in addition to those in 6.4 Lifting and Transferring the Patient, page 32 when transferring to or from a wheelchair:

1. Engage the wheel locks of the wheelchair to prevent movement of the wheelchair.
2. Position the patient over the seat with their back against the back of the chair.
3. Begin to lower the patient either by opening the control valve or by pressing the **DOWN** (▼) button.
4. With one assistant behind the chair and the other operating the patient lift, the assistant behind the chair will pull back on the grab handle (on select models) or sides of the sling to seat the patient well into the back of the chair. This will maintain a good center of balance and prevent the chair from tipping forward.

Use the straps or handles on the side and the back of the sling to guide the patient's hips as far back as possible into the seat for proper positioning.

5. Leave the sling in place.

Remove only if a divided leg sling was used.

7 Troubleshooting

7.1 Troubleshooting Table

SYMPTOMS	FAULTS	SOLUTION
Patient lift feels loose.	Mast/Base joint loose.	Refer to 4.2 Assembling the Mast to the Base, page 13.
	Linkage rods are loose.	Refer to 8.14 Maintaining the Base Adjustment, page 47.
Casters/Brakes noisy or stiff.	Fluff or debris in bearings.	Refer to 8.16 Replacing Front Casters, page 48 and 8.15 Replacing Rear Casters, page 48.
Noisy or dry sound from pivots.	Needs lubrication.	Refer to 8.3 Lubricating the Lift, page 40.
Oil leaking from hydraulics.	Hydraulic pump in need of replacement.	Refer to 8.8 Replacing a Manual/Hydraulic Pump, page 41. Contact your Dealer.
Manual / Hydraulic pump fails to lift when pumped.	Control Valve not fully closed.	Close Control Valve.
	Manual / Hydraulic pump in need of replacement.	Refer to 8.8 Replacing a Manual/Hydraulic Pump, page 41. Contact your Dealer.
		Check connections.
	Control valve may be out of adjustment.	Refer to 8.9 Adjusting the Pump Control Valve, page 42.

SYMPTOMS	FAULTS	SOLUTION
Electric actuator fails to lift or legs fail to open when button is pressed.	Hand-control or actuator connector loose.	Connect hand control or actuator connector. Ensure connectors are seated properly and fully connected.
	Battery low.	Charge batteries. Refer to 5.7 Charging the Battery, page 24.
	RED emergency stop button pressed IN.	Rotate RED emergency stop button CLOCKWISE until it pops out.
	Battery not connected properly to control box.	Reconnect the battery to the control box. Refer to 5.7 Charging the Battery, page 24.
	The connecting terminals are damaged.	Replace the battery pack. Refer to 5.7 Charging the Battery, page 24.
	Boom or leg actuator in need of service or load is too high.	Refer to 8.10 Replacing the Electric Actuator, page 44. or 8.11 Replacing the Leg Actuator, page 45. Contact your Dealer.
Unusual noise from actuator.	Actuator is worn or damaged or spindle is bent.	Refer to 8.8 Replacing a Manual/Hydraulic Pump, page 41, 8.10 Replacing the Electric Actuator, page 44 or 8.11 Replacing the Leg Actuator, page 45. Contact your Dealer.
Boom will not lower in uppermost position.	Boom requires a minimum weight load to lower from the uppermost position.	Pull down slightly on the boom.
Boom will not lower during a power retraction.	Shoulder bolt at the junction of the boom and mast may not be properly installed.	Refer to 8.12 Checking and Tightening Mast Pivot Bolt, page 46.
Hydraulic pump drifts when under load.	Control valve may be out of adjustment.	Refer to 8.9 Adjusting the Pump Control Valve, page 42.

If problems are not remedied by the suggested means, please contact your dealer or Invacare.

8 Maintenance

8.1 Safe Maintenance

**WARNING!****Risk of Falling**

Maintenance **MUST** be performed only by qualified personnel.

Improper assembly may cause injury or damage.

- Regular maintenance of patient lifts and accessories is necessary to assure proper operation.
- The pump is sealed at the factory. **DO NOT** attempt to open the pump or obtain local service as this will **VOID** the warranty and might result in damage. Consult your dealer or write Invacare for further information.
- **DO NOT** overtighten the mounting hardware. This will damage the mounting bracket.

8.2 Maintenance Safety Inspection Checklist

Regular cleaning will reveal loose or worn parts, enhance smooth operation and extend the life expectancy of the lift.

Follow the maintenance procedures described in this manual to keep your patient lift in continuous service.

The Invacare Patient Lift is designed to provide a maximum of safe, efficient and satisfactory service with minimum care and maintenance.

All parts of the Invacare Lift are made of the best grades of steel, but metal to metal contact will wear after considerable use.

There is no adjustment or maintenance of either the casters or brakes, other than cleaning, lubrication and checking axle and swivel bolts for tightness. Remove all debris, etc. from the wheel and swivel bearings. If any parts are worn, replace these parts immediately.


If you question the safety of any part of the lift, contact your Dealer immediately and advise him/her of your problem.

The items in the following checklist should be checked initially and in the following intervals:

- INSTITUTIONAL USE — Monthly and prior to each new user
- IN-HOME USE — Every Six Months

Date of Inspection:		Initials:
THE CASTER BASE ☐ Inspect for missing hardware. ☐ Base opens/closes with ease. ☐ Inspect casters and axle bolts for tightness. ☐ Inspect casters for smooth swivel and roll. ☐ Inspect and clear wheels of debris. ☐ Inspect pivot joints for wear.	THE MAST ☐ Mast MUST be securely assembled to boom. ☐ Inspect for bends or deflections. ☐ Inspect pivot joints for wear.	THE HANGER BAR ☐ Check the bolt/hooks for wear or damage. ☐ Check sling hooks for wear or deflection. ☐ Inspect pivot joints for wear. ☐ Check that strap retainers are in place and functioning.
SLINGS AND HARDWARE ☐ CHECK ALL SLING ATTACHMENTS each time it is used to ensure proper connection and patient safety. ☐ Inspect sling material for wear. ☐ Inspect straps for wear.	THE BOOM ☐ Check all hardware and hanger bar supports. ☐ Inspect for bends or deflections. ☐ Inspect bolted joints of boom for wear. ☐ Inspect to ensure that the boom is centered between the base legs. ☐ Check the mast pivot bolt. Ensure that the bolt is tightly secured. ☐ Inspect pivot joints for wear.	THE MANUAL/HYDRAULIC PUMP/ELECTRIC ACTUATOR ASSEMBLY ☐ Check for leakage. ☐ Inspect hardware on mast, boom and base. ☐ Check for wear or deterioration. IF DAMAGED, RETURN TO FACTORY. ☐ Cycle to ensure smooth quiet operation of the electric actuator. ☐ Check for smooth operation of the pump handle.
THE SHIFTER HANDLE ☐ Operates smoothly. ☐ Locks adjustable base whenever engaged.	THE CONTROL VALVE ☐ Opens and closes easily.	CLEANING ☐ Whenever necessary.

8.3 Lubricating the Lift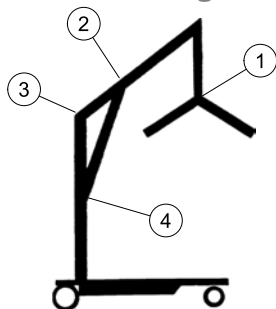

The Invacare lift is designed for minimum maintenance. However, a six month check and lubrication should ensure continued safety and reliability.

Keep lift and slings clean and in good working order. Any defect should be noted and reported to your Dealer as soon as possible.

Refer to the figure for lubrication points. Lubricate all pivot points with a light grease (waterproof auto lubricant). Wipe all excess lubricant from lift surface.

1. Swivel Bar
2. Boom Mounting Bracket
3. Boom/Mast Mount
4. Mast Mounting Bracket

8.4 Detecting Wear and Damage

It is important to inspect all stressed parts, such as slings, hanger bar and any pivot for slings for signs of cracking, fraying, deformation or deterioration. Replace any defective parts immediately and ensure that the lift is not used until repairs are made.

8.5 Cleaning the Sling and Lift


The sling should be regularly washed in water, temperature not to exceed 180°F (82°C), and a biocidal (anti-biological) solution. A soft cloth, dampened with water and a small amount of mild detergent, is all that is needed to clean the patient lift. The lift can be cleaned with non-abrasive cleaners.

8.6 Resetting the Service Light

This procedure should only be performed by a qualified technician.

This procedure should only be performed after performing 4.7 Checking the Service Light, page 18.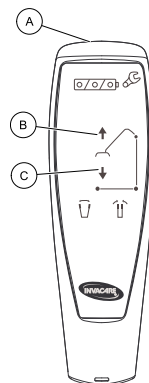

1. Locate the handset (A).
2. Press and hold the UP button (B) and DOWN button (C) at the same time for five seconds.

i You will hear a sound when the service light has been reset.

The service light is now reset to indicate a service requirement after a 12 month period or 8000 cycles.

If the service light is required to be set for a period shorter than 12 months, a special handset and further instructions are required. Contact Invacare to order this handset and instructions.

8.7 Maintaining the Manual/Hydraulic Pump


All parts of the Manual/Hydraulic Pump are precision machined, then carefully assembled and tested to ensure reliable service. The pump assembly is completely enclosed and sealed with neoprene rings to prevent leakage of hydraulic oil. A small amount of oil (about a drop) will accumulate around the piston from time to time and should be removed with a facial tissue.

WARNING!

- The pump is sealed at the factory. **DO NOT** attempt to open the pump or obtain local service as this will void the warranty and might result in damage and costly repair. Consult your dealer or write Invacare for further information.

8.8 Replacing a Manual/Hydraulic Pump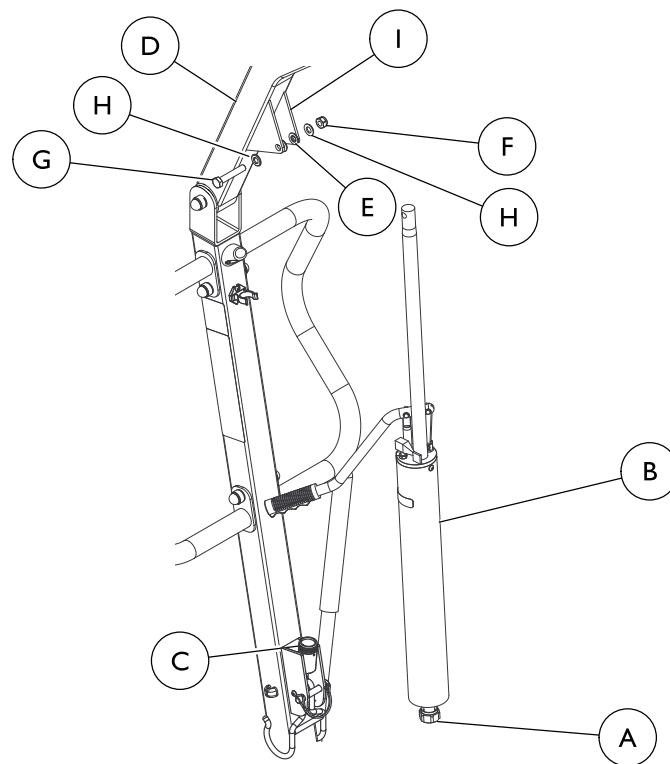


**CAUTION!****Risk of Damage**

Improper assembly may cause injury or damage.
– DO NOT overtighten the mounting hardware. This will damage the mounting bracket.

1. Loosen the nut (A) that secures the ball of the pump (B) into the mounting socket (C).
2. Rest the boom (D) on your shoulder and remove the nut (F), bolt (G), nylon washer (E) and washers (H) from the boom mounting bracket (I).
3. Remove the pump.
4. Reverse steps for installation.

8.9 Adjusting the Pump Control Valve

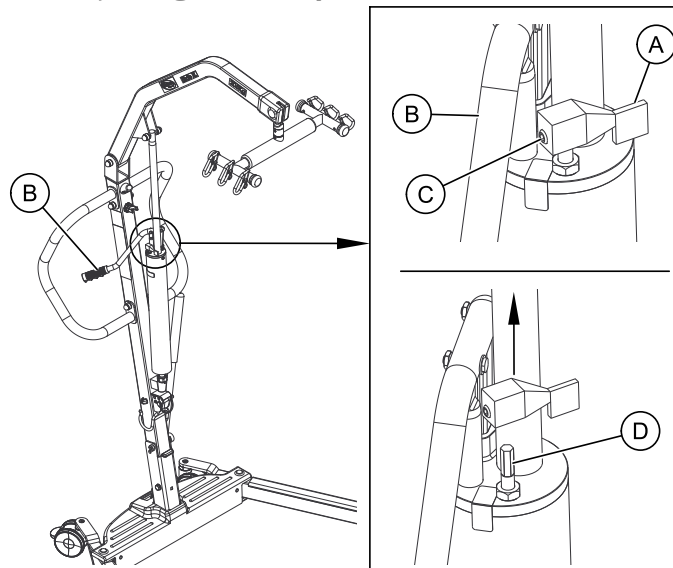**WARNING!**

– Always remove the patient from the lift before performing this procedure. DO NOT perform this adjustment while lifting a patient. Serious injury may occur.


This procedure applies to the ILIFTHM models only.

1. Turn the control valve (A) so it will clear the pump handle (B) when the handle is fully retracted.
2. Loosen the set screw (C) a few turns.

3. Remove the control valve lever.
4. Hand tighten the control rod Ⓒ by turning it clockwise.
5. Reinstall the control valve lever.
6. Position the lever so it clears the handle when it is fully retracted.
7. Tighten the set screw.
8. Test the hydraulic pump with a load.

8.10 Replacing the Electric Actuator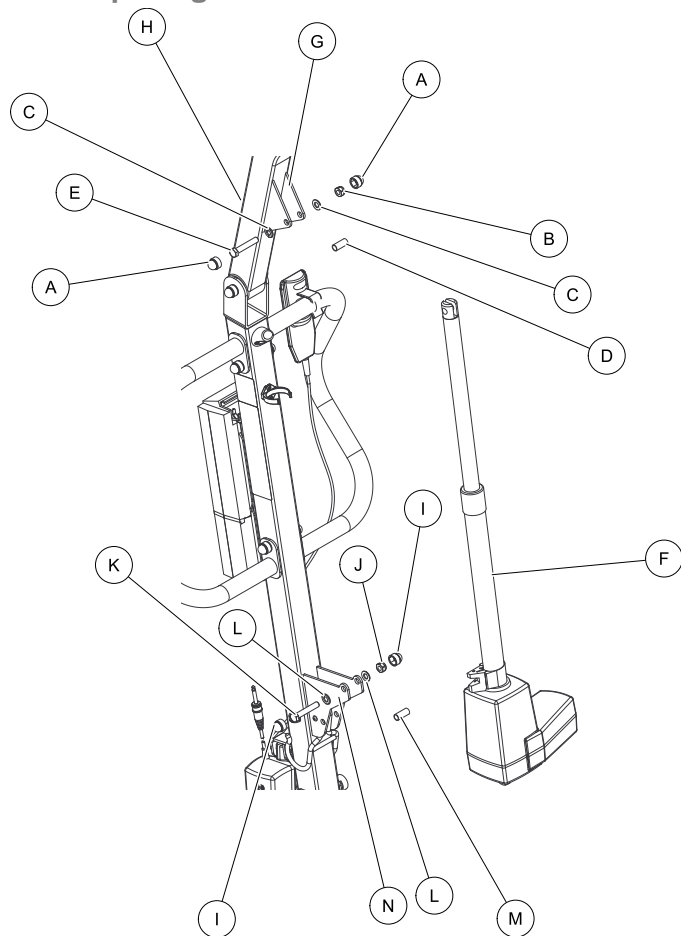

1. Remove the caps (A), nut (B), washers (C), bushing (D) and shoulder bolt (E) that secure the actuator (F) to the boom mounting bracket (G).
2. Rest the boom (H) on your shoulder and remove the caps (I), nut (J), bolt (K), washers (L) and bushing (M) from the mast mounting bracket (N).
3. Remove the electric actuator.
4. Reverse STEPS 1-3 for installation.

CAUTION!

Risk of Damage

Improper assembly may cause injury or damage.


– DO NOT overtighten the mounting hardware. This will damage the mounting bracket.


Be sure that the bolt or pin is completely through the holes of the boom assembly mounting bracket and the actuator assembly. The boom assembly will pivot easily if the mounting hardware is aligned properly when the boom assembly is secured to the mast.

8.11 Replacing the Leg Actuator

A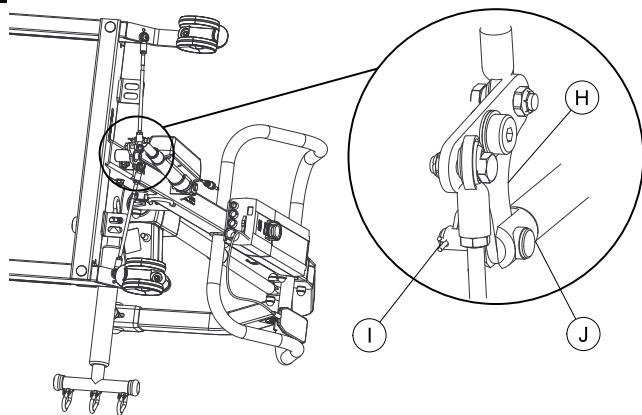

B

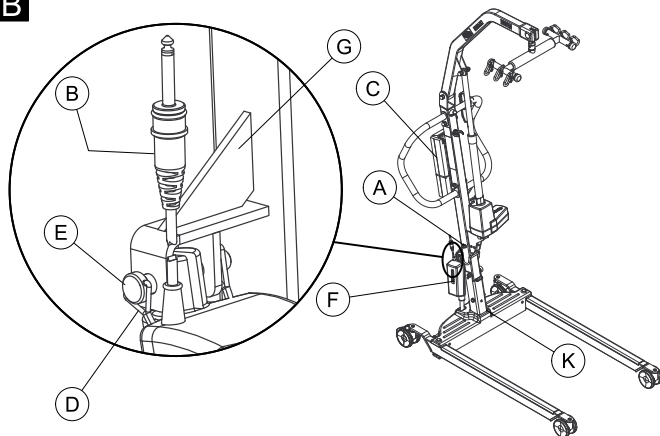1. Remove the wire lock (A) that secures the leg actuator cable to the mast.
2. Disconnect the leg actuator cable (B) from the bottom of the control box (C).
3. Remove hitch pins (D) securing the pin (E) to the leg actuator (F) and mast bracket (G).
4. Remove the pin from the leg actuator and mast bracket.
5. Perform the following to remove the leg actuator from the pivot bracket (H) (Detail A):

WARNING!

Risk of Injury

Injury to personnel or damage to the patient lift may occur.


- Ensure that there is sufficient room to turn the patient lift on its side and that the floor area is clear of debris.

- a. Turn the patient lift on its side.
 - b. Remove the hitch pin (I) securing the pin (J) to the pivot bracket.
 - c. Remove the pin from the leg actuator and pivot bracket.
 - d. Return the patient lift to the upright position.
6. Slide the leg actuator out of the slot (K) in the base of the patient lift (Detail B).
 7. Reverse STEPS 1- 5 to install the leg actuator.


Position leg actuator into the mast bracket and move the legs to align the holes in the leg actuator with the holes in the mast bracket.

8.12 Checking and Tightening Mast Pivot Bolt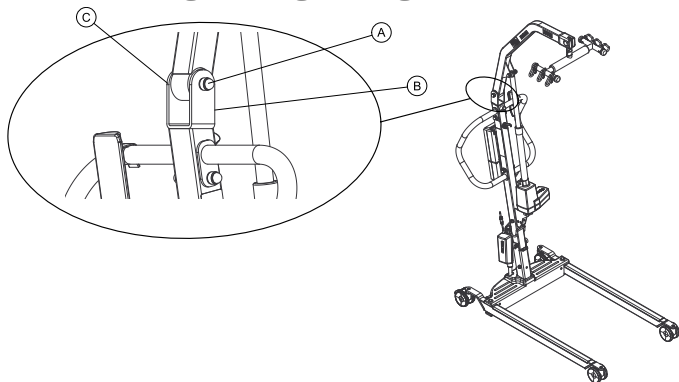

1. Check that the bolt (A) is through the bracket (B) and the locknut (C) is tight and secure.
2. If needed, do one or more of the following:
 - Tighten locknut and back-off the locknut 1/8 of a turn.
 - Replace the locknut.

8.13 Replacing the Swivel Bar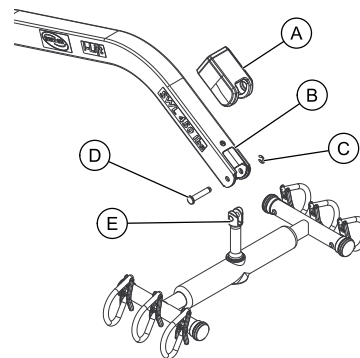

WARNING!

Risk of Injury

Worn or damaged parts of the lift may cause injury to the patient or assistants.


- After the first year of use, the hooks of the hanger bar and mounting brackets of the boom should be inspected every six months to determine the extent of wear. If these parts become worn, replacement must be made.

If the scale is installed on the lift, refer to 9.3 Installing the Scale, page 50 to replace the hanger bar.

1. Remove the rubber cover (A) from the boom (B).
2. Remove the clip (C) and clevis pin (D) securing the hanger bar pin (E) to the boom.
3. Reverse STEPS 1- 2 to install the new hanger bar.

8.14 Maintaining the Base Adjustment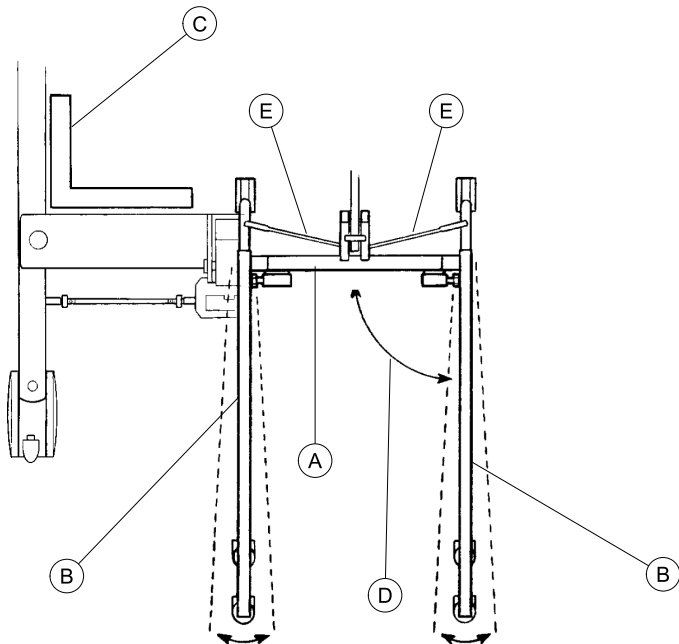

The base (A) adjustment should not require any attention other than adjusting the linkage rods.

Damage to the lift may occur.

– The leg actuator must be in the fully closed position before adjusting the linkage rods.


1. Close the legs all the way. Refer to Closing/Opening Manual Legs, page 19 or Closing/Opening Electric Legs, page 20.
2. Check the squareness of the legs (B) when in the closed position.
3. Place a square (C) on the inside of the legs and base to determine the 90° alignment (D).
4. Adjust the linkage rods (E) until 90° alignment is achieved.

Damage to the lift may occur.


– DO NOT adjust the linkage rods so the legs are at less than 90° alignment.

8.15 Replacing Rear Casters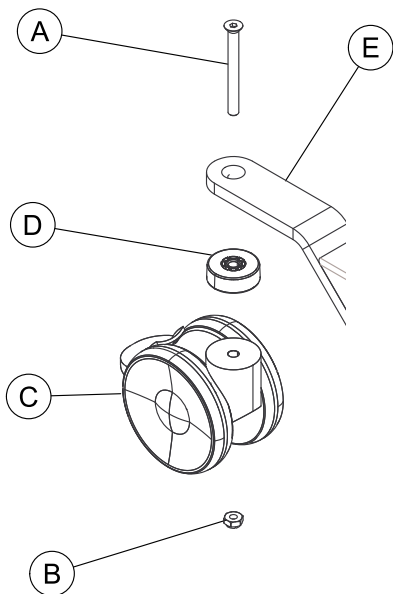

1. Place the lift on its side.
2. Remove the bolt (A) and locknut (B) that secure the existing rear caster (C) and bearing (D) to the rear caster bracket (E).
3. Install the bolt through the rear caster bracket, bearing and new rear caster and tighten securely with the locknut.

8.16 Replacing Front Casters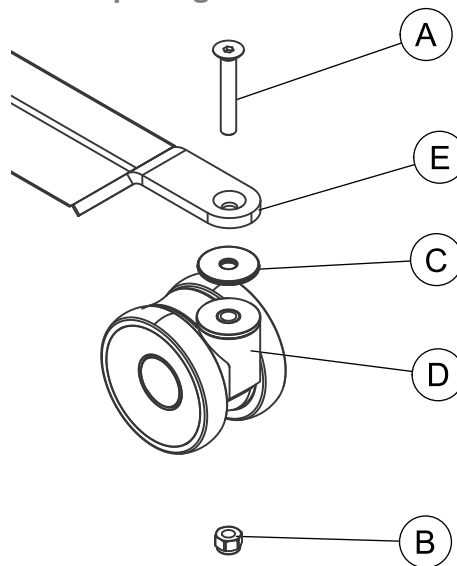

1. Place the lift on its side.
2. Remove the bolt (A), bearing (C) and locknut (B) that secure the existing front caster assembly (D) to the front caster bracket (E).
3. Install the bolt through the front caster bracket and the new front caster assembly and tighten securely with the locknut.

9 Scale

9.1 Scale Introduction

The I-Lift Scale is a compact precision scale system designed specifically for the Invacare Patient Lift System.


WARNING!

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

- Patient and sling **MUST** be removed from the lift during ALL installation procedures.

9.2 Removing the Swivel Bar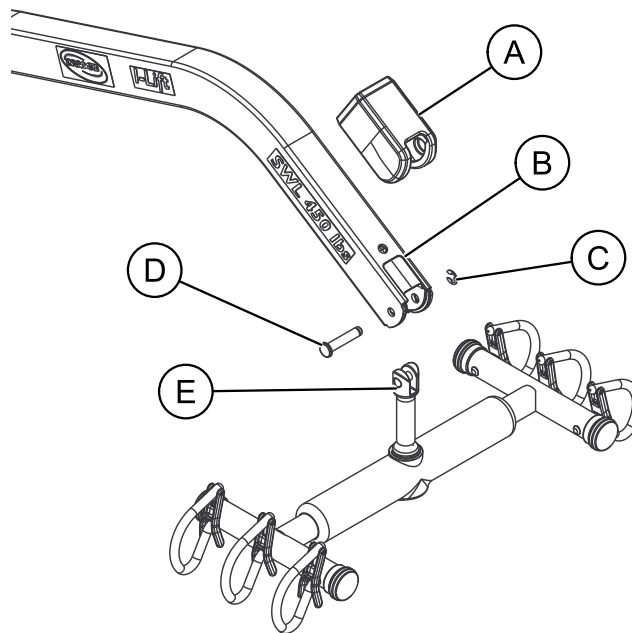


1. Remove the rubber cover (A) from the boom (B).
2. Remove the clip (C) and clevis pin (D) securing the hanger bar pin (E) to the boom.

Save the clip and clevis pin to secure the I-Lift Scale to the boom.

9.3 Installing the Scale

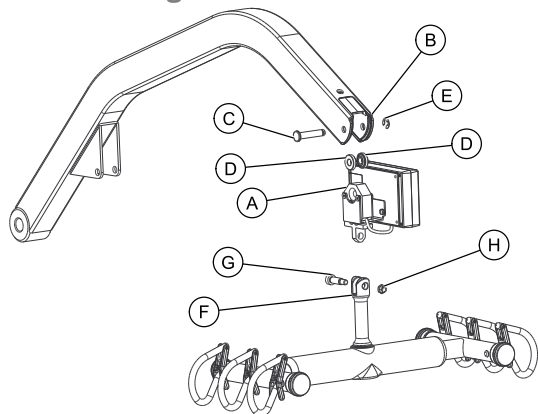1. Remove the hanger bar. Refer to 9.2 Removing the Swivel Bar, page 49.
2. Position the load cell assembly (A) of the scale into the boom mounting bracket (B).
3. Secure the scale to the boom mounting bracket with the clevis pin (C), two spacers (D) and the clip (E).
4. Secure the hanger bar (F) to the scale with the shoulder bolt (G) and locknut (H).
5. Weigh the patient. Refer to 9.5 Weighing the Patient, page 52.

9.4 Operating the Scale

Keypad Functions

Key	Indicator Displayed	Indicator Location	Definition
ON OFF	OFF	Center of Display Window	Pressing this key will apply power to the scale and turn the unit on. When the scale is already on, pressing the button will turn the unit off.
ZERO	ZERO	Lower Left Corner of the Display Window	Pressing this key when the scale is on will reset the weight shown in the display window to zero.
UNITS	lb or kg	Upper Right Corner of the Display Window	The function of this key is to change the unit of measurement from pounds (lb) to kilograms (kg).
LOCK UNLOCK	LOCK	Lower Right Corner of the Display Window	This key is used to lock or unlock a weight value in the display window. NOTE: Weight displayed is a STORED weight and not the current weight when the key is used in the lock position.
N/A	LO BAT	Center of Display Window	Indicator is shown in the display window to notify when battery is low.

9.5 Weighing the Patient

WARNING!

- The weight capacity is limited to the lowest rated capacity of any one of the components in use (e.g. Patient Lift, Sling or Scale). The patient's weight **MUST** not exceed the lowest rated capacity of any component.

1. Attach sling straps to the hanger bar. For proper attachment instructions refer to 6.3 Attaching the Slings to the Lift, page 30.

For improved accuracy on the I-Lift, attach all sling straps to the center hooks on both sides of the hanger bar. However, patient comfort may require a different configuration of sling straps.

2. Press the ON/OFF key.

The display will indicate the last weight that was measured. The word "LOCK" will be seen in the display box.

The ZERO key is pressed in order to avoid capturing the weight of the sling and the hardware. If the ZERO key is not pressed the weight of the sling and the weight of the hardware will be included in the weight displayed. Not zeroing out will give a false reading of the patient's true weight.

3. Press the ZERO key. When the ZERO key is pushed the following will happen:
 - a. The scale will reset to zero and the word "ZERO" will appear in the display.
 - b. The lock function will then be turned off and the word "LOCK" will disappear from the display.

The scale is now active and continually updating the weight display.

4. Place the patient in the sling.
5. Attach the sling to the lift. Refer to 6.3 Attaching the Slings to the Lift, page 30.
6. Activate the lift mechanism to raise the patient until they are completely supported by the lift. For patient lifting instructions, refer to 6.4 Lifting and Transferring the Patient, page 32.
7. Note the weight display.
8. When the weight display becomes stable press the LOCK button to lock the weight display. This will be indicated by the word "LOCK" appearing in the display window.


Should it be necessary to unlock the weight while the patient is still supported by the lift, the UNLOCK button may be pressed. The weight will unlock and the word "LOCK" will disappear from the display window. The weight value will then be updated. Once the weight becomes stable the weight can be locked again by pressing the LOCK button.

Stable being defined as the weight fluctuating two tenths of a pound. For example, a patient weighing one hundred pounds, the scale will fluctuate between 99.8 and 100.2 until the LOCK key is pressed. Fluctuation of the weight displayed is normal as noted above. Press the LOCK button to lock the weight.

9. The lift may now be lowered and the sling removed from the patient.

i The patient's weight will continue to be seen in the display window. The display will turn off automatically after a two minute period of non-use [no changes in weight exceeding five pounds (two kilograms)]. You can NOT adjust the time delay for automatic shut off. After the display has turned off, the weight may be recalled by pressing the ON/OFF button. The unit can be turned off by pressing the ON/OFF button a second time.

9.6 Replacing the Battery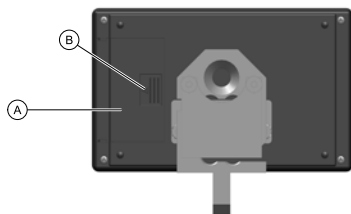


i The scale is powered by a nine volt alkaline battery that should provide approximately 1500 readings before needing replacement.

When battery replacement is needed, lo bat will appear on the display. Perform the following:

1. Slide the battery door (A) open in the direction of the arrow (B).
2. Remove existing battery.
3. Install the new battery.
4. Reinstall the battery door.

9.7 Calibrating the Scale

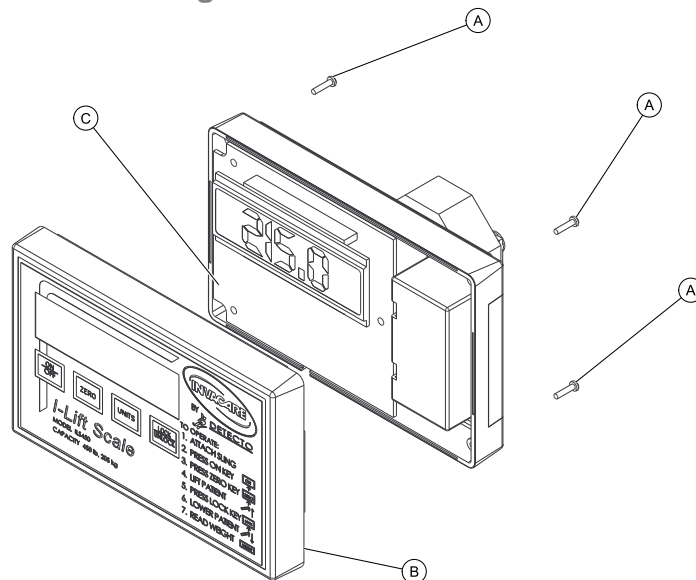i The I-Lift Scale will be pre-calibrated at the factory with the load cell. Should it be necessary to re-calibrate the scale, follow the instructions outlined below.

1. The patient and the sling must be removed from the scale to properly calibrate the I-Lift Scale. For removing the patient instructions refer to 6.4 Lifting and Transferring the Patient, page 32.
2. With the I-Lift Scale on, remove the four screws (A) on the back of the enclosure and remove the front cover (B) to expose the PC board (C).

3. Press the CAL button located on the PC board. The CAL switch is not labeled but is the only button on the PC board located in the lower right corner. Once the CAL button is pushed “CAL1” will be seen in the display window.

 CAL1 selects the calibration mode using 50 pounds of calibrated weight. CAL2 selects the calibration mode for use with 200 pounds of calibrated weight. Pressing the UNITS key toggles between CAL1 and CAL2.

4. When the desired calibration mode is displayed, press the LOCK/UNLOCK key.

 The display window will now show UnLd.

5. Ensure that there is no load on the scale and press the ZERO key.
6. The display window will show a dashed line (-----) scrolling across and then the word “LOAD” will appear in the display window.
7. Perform one of the following:
 - a. FOR CAL1 OPTION - support 50 pounds of calibrated weight from the scale and press the ZERO key.
 - b. FOR CAL2 OPTION - support 200 pounds of calibrated weight from the scale and press the ZERO key.
8. The display window will show a dashed line (-----) scrolling across and then the word “DONE” will appear in the display window.
9. Remove the weight from the scale and press the LOCK/UNLOCK key.

 The unit will now be in a temporary test mode and will not lock the weight display. This will allow weight to be loaded and unloaded to check the calibration.

10. Turn the unit off by pressing the ON/OFF key.
11. Turn the unit on by pressing the ON/OFF key.

 Scale will now be in normal operation.

9.8 Troubleshooting

SYMPTOM	PROBABLE CAUSE	SOLUTIONS
Unit does NOT work properly.	Battery failure.	Check battery. Replace if necessary.
Battery has been replaced and unit still does NOT work properly.		Contact Invacare for Service at 1-800-333-6900

Display Codes

	CALIBRATION REQUIRED - Indicates improper stored calibration data, calibration is necessary.
	OVER CAPACITY - Indicates a weight exceeding the capacity has been loaded on the scale.

10 Warranty

10.1 Limited Warranty — North America

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the products manufactured to be free from defects in materials and workmanship for a period of three years on the lift and one year on the slings, hydraulic pump / electric components from the date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, date of purchase, indicate nature of the defect.

Invacare Corporation will issue a serialized return authorization. The defective unit or parts **MUST** be returned for warranty inspection using the serial number, when applicable as identification within 30 days of return authorization date. Do not return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY

1171892-B

SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE INSTRUCTIONS IN THIS MANUAL.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSIONS AND LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE OR PROVINCIAL LAWS AND REQUIREMENTS.

II Survey

II.1 Usability Survey

Please complete the survey below to evaluate this manual.

Your participation in the evaluation assists in the development of effective and usable manuals for our customers.

The survey is also available online:

<http://www.invacare.com/TechnicalDocumentSurvey>

Yes, you can!

1. Please indicate your primary involvement with the product (choose one):

- | | |
|---|--|
| ☐ Product User/Owner | ☐ Product Service Technician |
| ☐ User Assistant | ☐ Health Care Provider |
| ☐ Product Dealer | ☐ Other (please specify): _____ |

2. Please indicate which product manual you are evaluating:

3. Evaluate the content:

	YES	NO
After reading this document, do you have a better understanding of how to use the product?	☐	☐
Do you have a better understanding of any limitations on the use of this product?	☐	☐
Is there any irrelevant information?	☐	☐
Is the Table of Contents useful?	☐	☐
Does any information seem inaccurate/misleading?	☐	☐
Do you understand that misuse of the product can cause injury or damage?	☐	☐
Explain:	_____ _____	

4. Evaluate the Warnings/Cautions:

	YES	NO
Are there any warnings/cautions that you do not understand?	☐	☐

Are there too many warnings/cautions? ☐ ☐

Are there warnings/cautions that you feel do not apply to this product? ☐ ☐

Explain:

5. Evaluate the style:

	YES	NO
Is anything hard to locate/follow?	☐	☐
Are any headings missing/confusing?	☐	☐
Are there too many headings?	☐	☐
Should any material be a bulleted list or checklist instead of numbered steps or a paragraph?	☐	☐
Is there material that might be clarified by a visual?	☐	☐

Explain:

6. Evaluate the illustrations:

	YES	NO
Are the illustrations useful?	☐	☐
Do the illustrations need more or less detail?	☐	☐
Is the number/size of illustrations adequate?	☐	☐

Explain:

7. Do you have suggestions for other ways of making this document easier to use?

	YES	NO
Explain:	☐	☐

Thank You! Thank you for completing this survey. If you have any questions or we may be of assistance to you, please feel free to contact us.

Send your survey to Invacare Technical Writing Department: TechnicalWriting@invacare.com
or Invacare Corporation: One Invacare Way, Elyria, Ohio 44035 FAX: 440-329-6975

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Invacare Rehabilitation Equipment (Suzhou) Co. Ltd.