



270 Washington Street, Mount Vernon, New York 10553

Tel: 845-205-6004 Fax: 845-205-6005

[www.proactivemedical.com](http://www.proactivemedical.com)

855-BE-PROACTIVE (855-237-7622)

## **Alternating Pressure Pump & Bubble Pad User Manual**

### **Standard Features**

- ◆ Simple to operate.
- ◆ Portable and lightweight.
- ◆ Pad consists of 130 bubble cells.

### **Operating Instructions**

1. Position pump on a horizontal surface or hang it by means of the hook.
2. Open mattress on the bed. Secure it with end flaps or straps. Air tubes must be at foot end of bed.
3. Connect air feed tubes to air outlet on the mattress and pump, ensuring that the tubes are not kinked or twisted. Allow space for bed rails to drop freely.
4. Plug the pump into a suitable electrical power source and switch on. The switch indicator should now illuminate.
5. Allow 20 minutes for mattress to inflate fully.
6. Cover with a sheet and tuck loosely.
7. Please use pressure adjustable knob to give maximum patient comfort.

### **Technical Specifications**

<b>Pump</b>	<b>Mattress</b>
Item: Protekt Aire 1000	Item: Protekt Aire 1000
Power supply: 110V/60Hz	Size: 110”(L) x 35.5”(W) x 2.6”(H)
Air output: $\geq 7$ liter/min.	130 bubble cells
Pressure range: 40 mmHg – 100 mmHg	Thickness: 0.28 mm
Cycle time: 10 min.	Without vents and with end flaps
Power consumption: 9W	Material: PVC (EN-71, low toxic)
Fuse: 1A	Color: beige
Size: 12”(L) x 6”(W) x 3.6”(H)	Repair kit included
Weight: 3.1 lbs.	

### **Cleaning Guidelines**

- ◆ Pad can be cleaned by disinfectant solution.
- ◆ Do not immerse the pump in water.
- ◆ Do not heat or steam in autoclave.

**Warranty for Pump and Mattress: 1 year**